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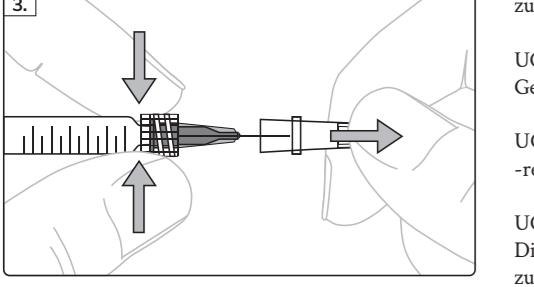
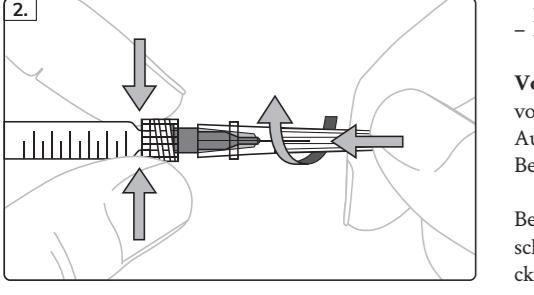
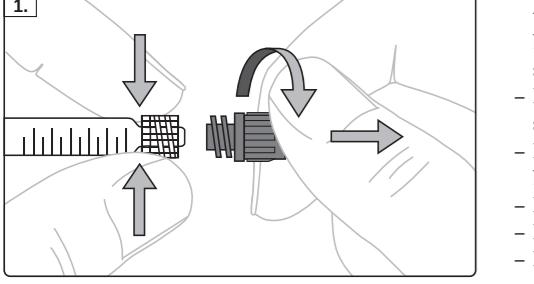
Side effects: The patient must be informed that they are possible side effects, which may occur immediately or may be delayed. These include, but are not limited to:

- Slight bleeding during the treatment which spontaneously subsides as soon as the injection is completed.
- Inflammatory reactions usually associated with injections such as redness, oedema, erythema, swelling, pain or hyperthermia sometimes accompanied with itching, burning or paraesthesia. These reactions may last for 3-5 days.
- Delayed injection-related inflammation such as persistent erythema or acne papule formation.
- Bleeding, haematoma or telangiectasia in the treated area.
- Swelling effect in other areas through displacement of adjacent structures or product migration.

Composition: UCderm volume contains:

- 26 mg/ml crosslinked Hyaluronic acid

- 3 mg/ml Lidocaine hydrochloride

- Phosphate buffer

The amount of gel in each syringe is 1 ml.

- 3 mg/ml Lidocaine hydrochloride

- Phosphate buffer

The amount of gel in each syringe is 1 ml.

Indications/Intended use: UCderm volume is a sterile, injectable, absorbable implant for augmenting the volume of soft facial soft tissues. This implant is indicated to lift different types of medium facial wrinkles, to fill and contour lips, to reduce scars and to balance anatomical or injury-related asymmetries. Moreover, it is also intended for correction of facial fat loss caused by illness, medication or predisposition. The product is intended to be used for injection into the mid to deep dermis or in the mucous membrane of the lips by a legally qualified, medical practitioner.

For successful treatment it is essential that the practitioner has been trained in injection technique for soft tissue augmentation. He/she has to be knowledgeable about the anatomy and physiology of the areas to be treated.

The lidocaine contained in the product is meant to reduce the patient's local pain during and for a short time after the injection.

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UCderm volume must not be injected into skin areas that contain residues of hyaluronic acid implants by other manufacturers. The last injection should be 3-6 months ago.

As a matter of general principle, injection of a medical device is associated with a risk of infection.

Patients should be instructed to report any side effects which last for more than one week to his/her practitioner. The practitioner may then prescribe appropriate treatment.

Warnings:

- Check the integrity of the package and the expiry date prior to use. Do not use the product if the package has already been opened or is visibly damaged or expiration date has been exceeded. UCderm volume has a shelf life of 3 years after the date of manufacture.

- Do not re-sterilise the product.

- Do not re-use the product.

- Used and partially used syringes and injection needles should be handled and disposed in accordance with facility medical practices and applicable regulations.

- In case of acute systemic infections.

- In case of skin inflammations (e.g. acne, herpes, psoriasis)

- In case of known hypersensitivity to sodium hyaluronate or lidocaine hydrochloride or amide-based local anaesthetics.

- In case of acute articular rheumatism with cardiac localization.

- In patients with a tendency to hypertrophic scarring.

- In case of untreated epilepsy.

- In patients who suffer from porphyria.

- During treatment with drugs tending to slow down or inhibit the liver metabolism (e.g. cimetidine, beta-blockers).

- In case of cardiac arrhythmia.

- Screw the supplied needle onto the syringe as far as it will go. Ensure not to twist the Luer lock.

- Keep holding the Luer-lock and carefully remove the protective cap of the needle.

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UCderm volume should be injected slowly and evenly. The amount to be injected depends on the correction to be achieved and the area to be treated. Do not overcorrect. The syringe has an indexing scale.

Storage: Store UCderm volume between 2°C and 25°C and protect it from freezing and direct sunlight. Avoid mechanical shocks.

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Never try to straighten a bent needle. Discard the bent needle and replace it.

If a needle appears dull, discard the blunt needle and use a new needle.

Gently massage the treated area after the injection to distribute the product uniformly.

Do not overcorrect.

The patient should avoid using any makeup for at least 12 hours after treatment as well as prolonged exposure to the sun, UV rays or temperatures below 0°C for at least 14 days after treatment. In addition, any sauna or hammam sessions should be avoided for up to two weeks after the injection.

The treated skin area has to heal completely first.

Consider that the product contains substances which may produce a positive result in anti-doping tests.

UCderm volume must not be injected into skin areas which had been subjected to a treatment with permanent implants, implants of animal origin or implants containing other substances than hyaluronic acid since this might trigger intolerances.

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